



# Dolphin Imaging

High Quality Digital Imaging Solutions for Dental Specialties

**WORLD HEADQUARTERS**  
6641 Independence Avenue  
Canoga Park, CA 91303, USA  
800.548.7241 • +1.818.888.0369  
+1.818.888.0370 FAX  
[www.dolphinimaging.com](http://www.dolphinimaging.com)  
[info@dolphinimaging.com](mailto:info@dolphinimaging.com)

## OLYMPUS E-10 Recommended Settings (rev. 12/13/00)

### Additional Components:

- **SunPak DX 8R Ring Flash**
- 62mm-52mm ring adaptor

### Camera Default Settings

- Auto focus: AF (MF optional)
- "M" mode: **640 11**
- "S" mode: 640
- (file format): SQ (HSQ if you have a very fast computer)
- WB (White Balance): 6500K
- ISO: 80
- BKT: Frame OFF  
+/-: 1/3
- Flash: +/-: 0
- (s): NORMAL
- (c): NORMAL
- (File format):  
Depending on the processor speed & memory capacity of your computer: Here are our recommendations:  
*P166-P266, 64MB: SQ 640x480, 1/8*  
*P300-P400, 128MB: SQ 1024x768, 1/8*  
*P500+, 256MB: SQ 1600x1200, 1/8*  
*P1000+, 256MB: HSQ 1600x1200, 1/2.7*
- Interval Time: OFF
- (drive): 1
- (sound): ON
- REC VIEW: AUTO
- SLEEP: 1 MIN
- Focus Ring: always leave it for Telephoto all the way--36

### Ring Flash:

- "M"
- DIN: ISO 100
- Power: BATT

### Notes:

- If use AF: frame the face, push shutter trigger half way for auto-focus, listen for auto-focus "beep" then push all the way to capture
- When using AF, please be sure you are at the *same* distances between frames. Otherwise, if you are a lot closer to the subject and the resultant picture will be very bright.

- The AF mechanism requires sufficient ambient light. If capturing photos within a dark environment with AF, pictures might be guaranteed to be focused consistently.
- If use MF: move in & out until subject is focus, push the shutter trigger to capture
- Wait at least 5-10 seconds between pictures for the flash to recharge sufficient. If the picture is captured too quickly, you'll get a very dark looking image.
- To conserve battery life, lightly touch the shutter button when you've confirmed the picture review on the LCD. This will blank the LCD, thus reduce the usage of the battery.

### Facials:

*Assumes patient is in front of **Dolphin-supplied Portrait Lightbox** (with a "2-sheet" filter, patient 1'-1.5' from box)*

- Macro mode OFF (NO flower)
- **S** (Speed priority) "**640**"

### Ring Flash: 1/2

### Intraorals

*When capture I/O photos, you'll need to back out a bit; DON'T crop all the way*

- Macro Mode ON (**flower**)
- **M** (Manual mode) Speed: "**640**" and Aperture: "**11**". These settings should already been set.

### Ring Flash: 1/16

### Dolphin.ini for automatic zooming

```
[Capture Sequence - Facial-Intraoral-Occlusal -]  
6=(Filename, dsc00001.jpg), (Scale, 110)  
7=(Filename, ?Next), (Scale, 110)  
5=(Filename, ?Next), (Scale, 110)  
11=(Filename, ?Next), (Scale, 125)  
12=(Filename, ?Next), (Scale, 125)  
10=(Filename, ?Next), (Scale, 125)  
8=(Filename, ?Next),(UpsideDown,1), (Scale, 125)  
9=(Filename, ?Next),(UpsideDown,1), (Scale, 125)
```